## Annex of Action<sup>3</sup> user's guide

### Spécific adjustments for Action<sup>3</sup> Junior W/C

#### 1. Hangers

**1.1. Removable and swing away** (photo 1) : Standard footrest supports are swing away during transfers and can be removed during transport.

Operate lever (A) by pushing sideways and swivelling towards outside or inside in case there is not enough space. To return to the initial position, align the footrest support it locks automatically.



To remove the footrest support, simply pull up after unlocking the assembly.

Reverse the procedure to reassemble, while still in the unlocked position.

**1.2. Height adjustable** (photo 1-2) : Remove the hexagonal screw (B) and adjust the footplate to the desired height, based upon preset holes. Align the two holes and reposition the hexagonal screw, then tighten the assembly without forcing.

**1.3. Indexed to 0° or 30°** (photo 2) : Remove the hexagonal screw (C) and position the indexed plate (D) to 30° on the vertical upright (preset hole). Align the two holes and reposition the hexagonal screw, tighten the assembly firmly.

**Note** : the Action<sup>3</sup> Junior always is delivered with hangers adjusted to 0°.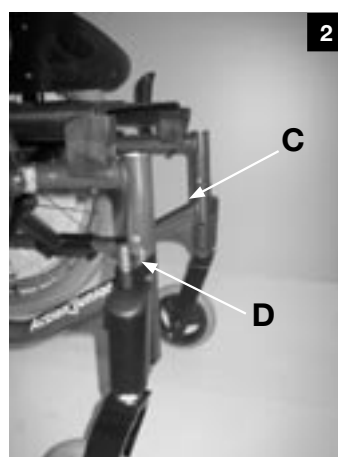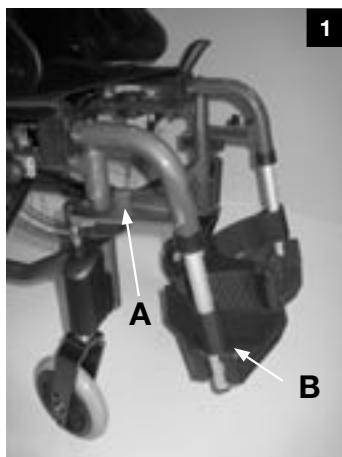

#### 2. Legrests

**Note** : the leg rest is specific to Action<sup>3</sup> Junior w/c and cannot be fitted on Action<sup>3</sup>.


**2.1. Removable and swing away** : To remove the leg rest, just pull it up. To swing it away, slightly pull up (about 2 cm) and swivel towards outside or inside.

**2.2. Adjustable** : (photo 3) Angle is adjustable through tooth plate : loosen the knob (A), adjust to the desired angle based upon notches and tighten firmly.

The calf pad is height adjustable by sliding : loosen the knob (B), adjust to the desired height, tighten firmly.

The footplate is height adjustable : loosen the knob (C), adjust to the desired height based upon the preset holes and tighten firmly.

The footplate is angle adjustable by tooth plate : loosen the knob (D), adjust to the desired angle based upon notches and tighten firmly.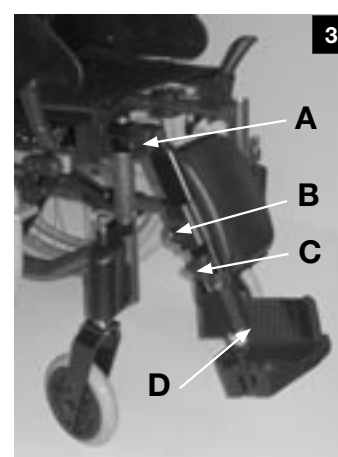

### 3. Armrests

**3.1. Simple** (photo 4) : To remove armrests, simply pull them up, reverse the procedure to reinstall them.

To adjust their height (3 positions), remove the screw (A), adjust to the desired height and reposition the screw (A), tighten without forcing.

Repeat previous procedure to remove.



**3.2. Multi adjustable** (photo 5) : Adding armrest height, depth and width adjustment; slightly loosen the ball screw (B), adjust to the desired height based upon the preset holes and tighten firmly; slightly loosen the two screws (C) and slide the armpad with its skirt-guard in for the desired depth or/and width, firmly tighten the two screws (C).

**Note:** the adjustment of the side protection is required based upon the desired height; slightly loosen the two mounting screws and slide the unit to the new position, then firmly tighten the two screws.

**Note:** the armrests are mounted in pairs on the wheelchair; whenever you remove them, remember that you have a right and a left side!

 Never lift the wheelchair by the armrests!

Keep your fingers away from movable parts during folding, disassembling or adjustment to prevent injuries!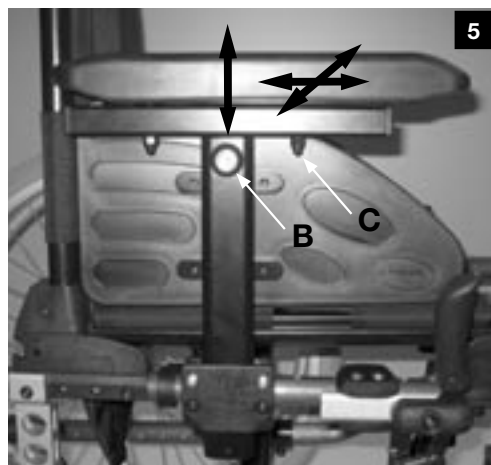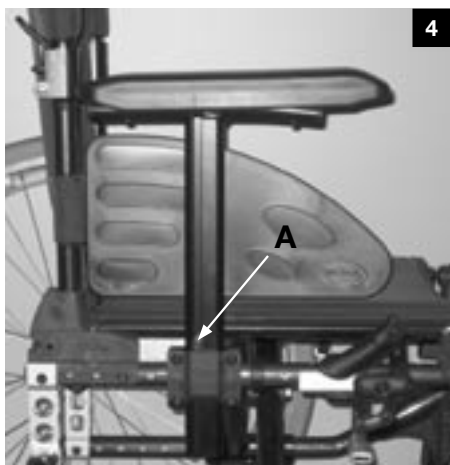

### 4. Headrest

Assembly (photo 6): Assemble the 2 clamps and the head-rest mechanism on the back brace by using the 2 provided screws (E), tighten firmly (screwdriver Pozidriv n°2) in the desired position.

To swing away for transfer, unscrew slightly the lever (A) and move backwards the headrest.

Reverse the operation to reinstall.


Adjust the angle and height of the whole headrest by loosening the lever (A) and the screws (B).

Then set the desired position and retighten firmly.

Adjust the angle of the "wings" (C) of the headrest by unzipping the zip at the headrest attachment and loosening the screw (D).

Set the required angle and close the zip.

Do not adjust the headrest when the user leans on it and check the mounting to the backrest in order to prevent injuries.

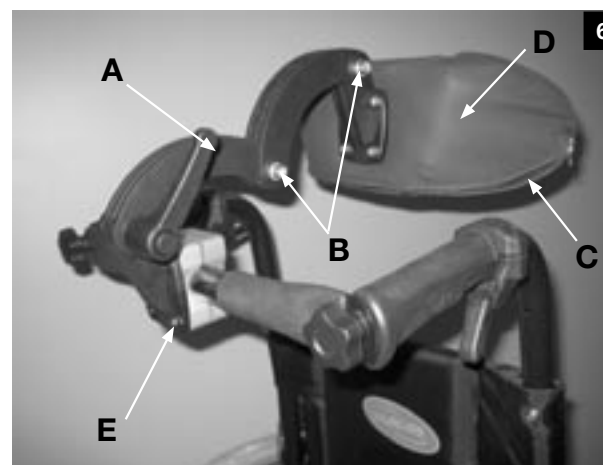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