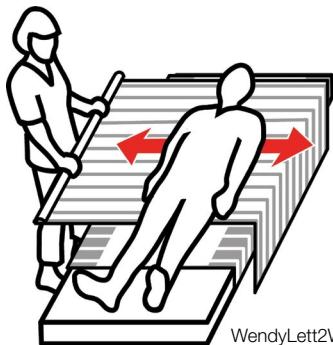
# WendyLett2Way

SystemRoMedic™


# WendyLett4Way

SystemRoMedic™

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WendyLett2Way

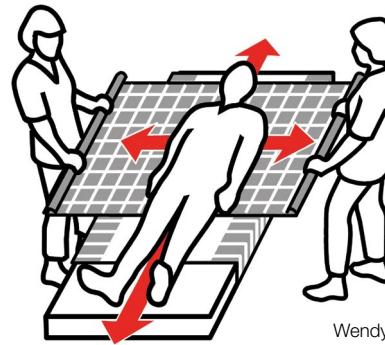WendyLett4Way

## REF

1620	1631
1636	1637
1647	1648
1649	1657

**SWL: 300 kg/660 lbs****Art.no. 1649 och 1657:****200 kg/440 lbs**

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### WendyLett - sliding sheet

(Can be combined with WendyLett2Way drawsheet or WendyLett4Way sliding drawsheet)

1609	WendyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	100 x 200/39 x 79	White
1614	WendyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	140 x 200/55 x 79	White
1629	WendyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	100 x 200/39 x 79	grey/white striped
1634	WendyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	140 x 200/55 x 79	grey/white striped
1639	WendyLett, 70 cm/28" sliding surface (bed 105 cm/42")	200 x 200/79 x 79	grey/white striped
1667	WendyLett, 100 cm/40" sliding surface (bed 120 cm/48")	100 x 220/39 x 88	grey/white striped
1668	WendyLett, 100 cm/40" sliding surface (bed 120 cm/48")	140 x 220/55 x 88	grey/white striped
1669	WendyLett, 100 cm/40" sliding surface (bed 120 cm/48")	200 x 220/79 x 88	grey/white striped
1641	WendyLett, 70 cm/28" sliding surface (bed 90 cm/36"), tailored with elastic spring	200 x 90/79 x 36	grey/white striped
1651	WendyLett, 80 cm/32" sliding surface (bed 105 cm/42"), tailored with elastic spring	200 x 105/79 x 42	grey/white striped
1661	WendyLett, 95 cm/37" sliding surface (bed 120 cm/48"), tailored with elastic spring	200 x 120/79 x 47	grey/white striped
1671	WendyLett, 110 cm/43" sliding surface (bed 140 cm/56"), tailored with elastic spring	200 x 140/79 x 55	grey/white striped

### WendyLett2Way - drawsheet

(To be combined with WendyLett sliding sheet)

1620	WendyLett2Way	90 x 180/35 x 70	white
1631	WendyLett2Way	100 x 200/39 x 79	white/grey striped
1636	WendyLett2Way	140 x 200/55 x 79	white/grey striped

### WendyLett4Way - drawsheet

(To be combined with WendyLett sliding sheet)

1637	WendyLett4Way	140 x 200/55 x 79	grey/white checked
1647	WendyLett4Way	200 x 200/79 x 79	grey/white checked
1648	WendyLett4Way, inco	75 x 200/30 x 79	grey (inco) checked
1649	WendyLett4Way, inco with handles	140 x 120/55 x 47	grey (inco)
1657	WendyLett4Way, with handles	140 x 190/55 x 75	grey/white checked
1652	WendyLett4Way	140 x 260/56 x 102	grey/white checked

### WendyLean - pillowcase with handles

1680	WendyLean	50 x 60/20 x 24	grey/white striped
------	-----------	-----------------	--------------------

# SystemRoMedic™

## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

### Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

**MediCauce**

MediCauce Soluciones Sanitarias SL  
C/ Rio Jarama 132, 10-13  
45007, Toledo. España  
T- (+34) 925 337 467  
[www.medicauce.com](http://www.medicauce.com)  
[info@medicauce.com](mailto:info@medicauce.com)